



APPLICATION INSTRUCTIONS

- ONE SIZE FITS ALL
- IMMEDIATELY STOPS BLEEDING
- APPLIES A SAFE PRESSURE— EVERY TIME
- HIGHLY VISIBLE—WON'T FORGET ON DIGIT

- ⊘ Do not use if package is damaged
- ⊘ Application not to exceed 90 minutes

PRECISION MEDICAL DEVICES LLC. SAN CLEMENTE, CA. - USA







SINGLE USE ONLY

Slide onto digit.

Bleeding stopped.
Perform the procedure.

Slide T-RING off digit, OR snap the T-RING in half by pressing against tabs, stretch & remove.

OR, Snap the T-RING and then cut to remove.

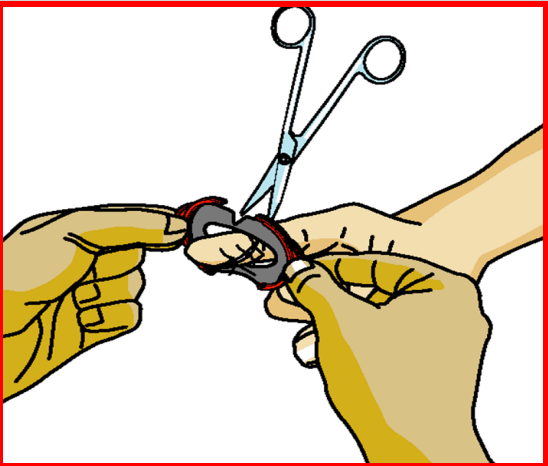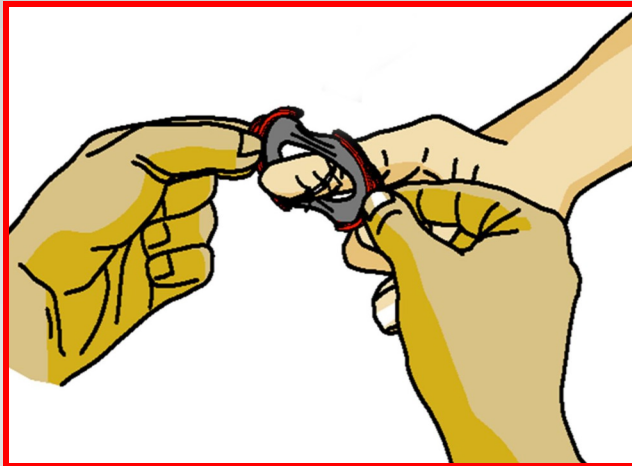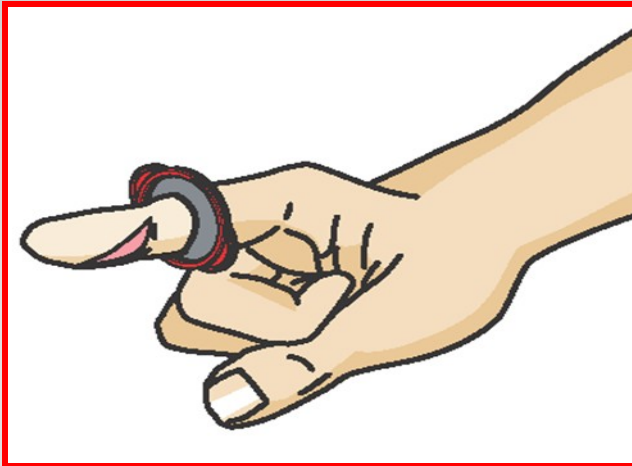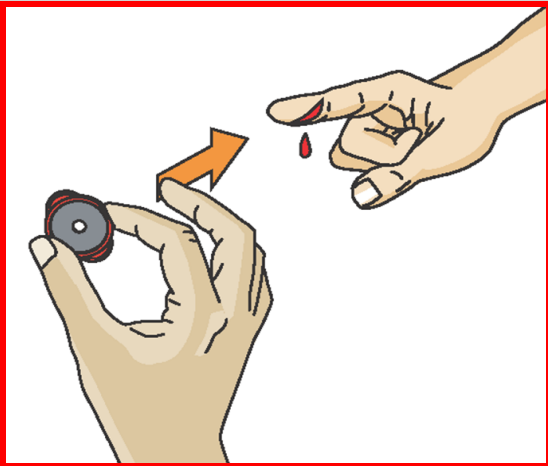

1

2

3



4

APPLICATION FOR LARGE FLAPS OR DEFORMITIES

Snap the T-RING before application by firmly holding tabs below, at the 3 and 9 O'clock positions

Firmly snap the T-RING at the 12 and 6 O'clock positions, (the break points) as seen below.

Stretch to enlarge opening until past wound, then slide to base of digit. Stretch or cut to remove.

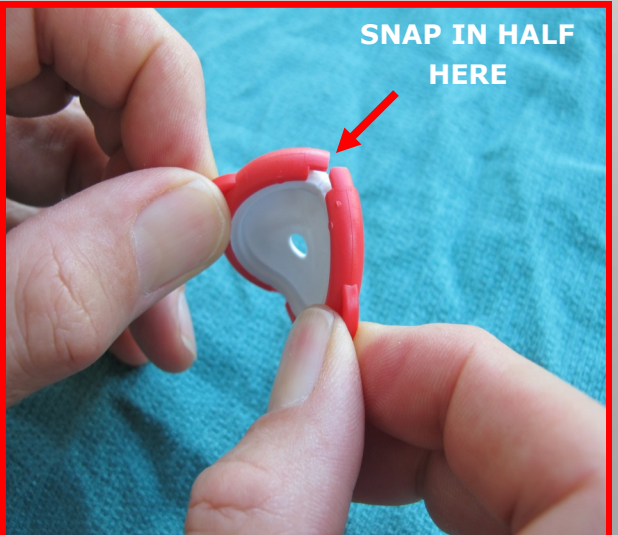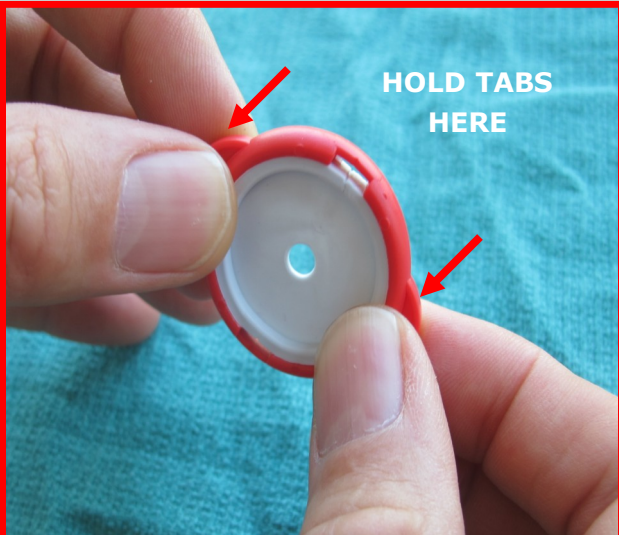

1

2

3