


# Novo-Cut

Single Patient Use Device



## Disposable Razor with Stainless Steel Blade

**Novo-Cut** is a medical disposable razor. With its platinum coated Swedish stainless steel razor blade it guarantees a clean shaving of the hair to be removed. Due to its handy design the razor easily follows the natural contours of the body without causing any skin irritations or cuts. The blue comb close to the blade effectively prevents a clogging of the blade by already removed hair. The viewing window allows a continuous monitoring of the part to be shaved. **Novo-Cut** is particularly suitable for dry shaving.

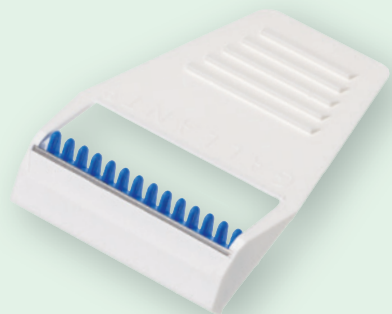

**Novo-Cut** is available as follows:

**Novo-Cut** REF 20 - 62      5,8 cm length    4,7 cm width

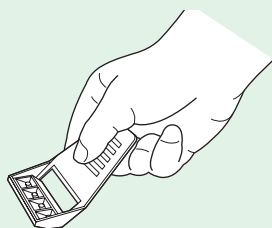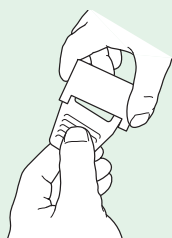

### INSTRUCTION FOR USE - APPLICATION

1. Remove the protective wrapping of the disposable razor.
2. Hold the razor by its fluted plastic handle and guide the stainless steel blade without pressure directly along the skin.
3. The razor has to be discarded after use. Please snap off the blade part of the razor. The razor breaks into two pieces at the predetermined breaking point.
4. Throw the separate blade into the disposal box for sharp and pointed items. Please also discard the plastic handle into the intended refuse container.



= do not reuse



= keep away from sunlight



= keep dry



= consult instruction for use

